AESCULIN BILE TEST

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Rev.2 / 26.05.2008

AESCULIN BILE TEST

Rapid test to evidence aesculin hydrolysis

DESCRIPTION

AESCULIN BILE TEST is a rapid test to identify Group D streptococci.

CONTENT OF THE PACKAGES

Each package contains 30 tubes with desiccated medium and one instruction sheet.

ITEMS NECESSARY BUT NOT INCLUDED IN THE PACKAGES

- PHYSIOLOGICAL SOLUTION (code 20095).
- Sundry microbiology laboratory materials.

PRINCIPLE OF THE METHOD

AESCULIN BILE TEST contains a selective and diagnostic medium for the detection of group D streptococci. These microorganisms grow in the presence of sodium azide and are capable of hydrolysing aesculin to glucose and aglycone 6-7 dihydroxycumarin: the aglycon reacts with the iron salts in the medium, giving a black colouring.

COMPOSITION

Each tube contains desiccated proteic substrata, iron salts, sodium azide and aesculin.

TEST PROCEDURE

- Take the package from the fridge, take one or more tubes and leave it for a few minutes on the bench until it has reached room temperature.
- Add 0.3 mL of PHYSIOLOGICAL SOLUTION (code 20095) to the tubes.
- With a sterile loop suspend in the culture medium of the tube a well isolated bacterial colony of 24 hours incubation chosen from the culture medium.
- Put the top to the tubes and incubate them at 36 ± 1 °C for 18 hours till a maximum of 24 hours.

INTERPRETATION OF RESULTS

Interpret the results according to the table 1:

Table 1

Colour of the medium	AESCULIN BILE TEST
Black	Positive
Colorless	Negative

QUALITY CONTROL FOR THE USER

1. Appearance: colorless desiccated medium adhering to the tube base.
2. Microbiological control.

Every batch of **AESCULIN BILE TEST** is subjected to the quality control, using one bacterial culture of *Staphylococcus aureus* ATCC 25923 as negative control and one of *Enterococcus faecalis* ATCC 29212 as positive control.

PRECAUTIONS

The product, **AESCULIN BILE TEST**, is not classified as hazardous under current legislation, neither does it contain noxious substances in concentrations $\geq 1\%$. **AESCULIN BILE TEST** is a disposable device to be used only for diagnostic use *in vitro*. It is intended for use in a professional environment and must be used in the laboratory by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

DISPOSAL OF USED MATERIAL

After the use, **AESCULIN BILE TEST** and the material that has come into contact with the sample must be decontaminated and disposed of in accordance with the laboratory procedures for the decontamination and disposal of potentially infected material.

STORAGE

Store **AESCULIN BILE TEST** at 2-8 °C in its original package. In such conditions the product is valid until the expiry date shown on the label. Do not use them beyond that date. Dispose of them if they show signs of deterioration.

REFERENCES

1. Isenberg, H.D., D. Goldberg, and J. Sampson. (1970). Appl. Micro., 20, 433-436.
2. NCCLS document M22-A2, 1996. Approved Standard.

PRESENTATION

Product	REF	
AESCULIN BILE TEST	88008	30 tubes

TABLE OF SYMBOLS

 In Vitro Diagnostic Medical Device	 Do not reuse	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	 Batch code

LIOFILCHEM s.r.l.

Via Scozia Zona Ind.le - 64026 Roseto D.A. (TE) - Italy

Tel. +390858930745 Fax +390858930330 Website: www.liofilchem.net E-Mail: liofilchem@liofilchem.net

Rev.2 / 26.05.2008