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### DESCRIPTION

**CATALASE/ OXY TEST** is a rapid test for the detection of catalase and peroxidase enzymes used in microbiology for the differentiation and identification of bacterial strains.

### CONTENT OF THE PACKAGES

Each package contains:

- 30 tubes containing desiccated Chlorpromazine
- 2 vials (2.5 mL) of H<sub>2</sub>O<sub>2</sub> Reagent
- 1 instruction sheet

### ITEMS NECESSARY NOT INCLUDED IN THE PACKAGES

- Physiological Solution (ref. 20095)

### PRINCIPLE OF THE METHOD

Chlorpromazine is a molecule constituted by an aliphatic chain which represents the substratum for enzymatic activity of catalase and peroxidase.

### COMPOSITION

(g/tube)

Chlorpromazine	0.1
----------------	-----

### USE

- Take one tube of **CATALASE/ OXY TEST** from the package and let it reach room temperature.
- Introduce 4 drops of Physiological Solution (ref. 20095).
- Inoculate using a sterile loop a well-isolated colony from an agar plate culture medium. Gently mix until all the substratum stuck to the walls of the tube is completely dissolved. Wait for 4-5 minutes.
- Add 3 drops of H<sub>2</sub>O<sub>2</sub> Reagent.

### INTERPRETATION OF RESULTS

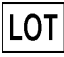
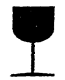
- Observe the turning of the color of the suspension to pink-red in one minute (PEROXIDASE TEST).
- Observe the development of bubbles (foam) on the surface of the suspension (CATALASE TEST).
- Identify the microorganism under examination using the reaction table.

### REACTION TABLE

PEROXIDASE TEST	RESULT
Pink- red color of the suspension	+
No turning in color	-

CATALASE TEST	RESULT
Bubbles development (foam)	+
No bubbles development	-

### TABLES OF SYMBOLS

 In Vitro Diagnostic Medical Device	 Use by	 Do not reuse	 Batch code	 Manufacturer
 Catalogue number	 Temperature limitation	 Fragile, handle with care	 Contains sufficient for <n> tests	 Caution, consult accompanying documents

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### QUALITY CONTROL

Each batch of **CATALASE/ OXY TEST** is submitted to the quality control using the following microorganisms:

Microorganism		Red color	Bubbles
<i>Escherichia coli</i>	ATCC 25922	-	-
<i>Staphylococcus aureus</i>	ATCC 25923	-	+
<i>Listeria monocytogenes</i>	CIP 7836	-	+
<i>Pseudomonas aeruginosa</i>	ATCC 27853	+	+

### PRECAUTIONS

**CATALASE/ OXY TEST** is classified as being hazardous according to the current legislation; see the safety data sheet for a correct use.

**CATALASE/ OXY TEST** is a disposable device to be used only for diagnostic use *in vitro*. It must be used in the laboratory by properly trained personnel, using approved aseptic and safety methods for handling pathogenic agents.

### STORAGE

Store **CATALASE/ OXY TEST** at 2-8°C in the original packaging. Keep away from sources of heat and avoid excessive changes in temperature. In such conditions, **CATALASE/ OXY TEST** will remain valid until the expiry date indicated on the label. Do not use beyond that date. Eliminate without using if there are signs of deterioration.

### DISPOSAL OF USED MATERIAL

After use, **CATALASE/ OXY TEST** and material that has come into contact with the sample must be decontaminated and disposed of in accordance with the techniques used in the laboratory for decontamination and disposal of potentially infected material.

### BIBLIOGRAPHY

- Galeazzi L., Grilli G., Turchetti G., Groppa G., Giunta S.: High Reactivity of the Chlorpromazine/Hydrogen Peroxidase Test with Cytocrome Oxydase-Positive Bacteria. Fems Microbiology Letters 56 (1988) 317-320. Pub. By Elsevier.
- Murray, Baron, Pfaller, Tenorev and Tenover: Manual of Clinical Microbiology. Current Edition.
- Edwin, H.Lenette.: Manual of Clinical Microbiology. Current Edition (ASM).

### PRESENTATION

Product	REF	Σ
<b>CATALASE/ OXY TEST</b>	<b>88023</b>	<b>30</b>

F00018  
Rev.3 / 21.10.2014

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**LIOFILCHEM® S.r.l.**