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Citrate Test

ENGLISH

For the identification of Enterobacteriaceae and non-fermenting gram-negative bacteria.

DESCRIPTION

Citrate Test is used in the identification of Enterobacteriaceae and non-fermenting gram-negative bacteria.

CONTENT OF THE PACKAGES

Each package contains 30 tubes and an instruction sheet.

PRINCIPLE OF THE METHOD

This test is useful in the differentiation of Enterobacteriaceae on the basis of the ability of some bacterial strains to use citrate as a carbon source. Bacteria which are capable of using citrate as their sole carbon source create an alkaline environment in the tube. At pH 7.5 or above, the pH indicator, bromthymol blue, turns blue while at a neutral pH it is green.

COMPOSITION

Citrate Test is a lyophilized mixture containing an ammonium salt as the sole nitrogen source, citrate as the sole carbon source, and a bromthymol blue indicator.

TEST PROCEDURE

1. Prepare a bacterial suspension (at least McFarland 0.5) of the strain to be tested in 0.25 ml physiological solution.
2. Add the entire microbial suspension to a Citrate Test tube.
3. Incubate at 36±1°C for 18-24 hours (positive reactions can sometimes be observed after 4-6 hours incubation).
4. Watch for the color changing of the suspension.

INTERPRETATION OF THE RESULTS

Citrate utilization (positive reaction) is indicated by the suspension turning blue. No citrate utilization (negative reaction) is indicated by the green color of the suspension.

QUALITY CONTROL FOR THE USER

Appearance: an orange lyophilized button per tube.

Microbiological control: cultural response observed on microbial suspensions added to Citrate Test tubes, after incubation at 36±1°C for 18-24 hours

Microorganism		Citrate reaction
<i>Klebsiella pneumoniae</i>	ATCC® 13883	+
<i>Proteus mirabilis</i>	ATCC® 25933	+
<i>Salmonella typhimurium</i>	ATCC® 14028	+
<i>Enterobacter cloacae</i>	ATCC® 23355	+
<i>Escherichia coli</i>	ATCC® 25922	-

PRECAUTIONS

The product does not contain hazardous substances in concentrations exceeding the limits set by current legislation and therefore is not classified as dangerous. It is nevertheless recommended to consult the safety data sheet for its correct use. Each tube is a disposable device intended for *in vitro* diagnostic use only and must be used in a professional environment by properly trained personnel, using approved asepsis and safety methods for handling pathogenic agents.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, does not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

DISPOSAL OF USED MATERIAL

After the use, Citrate Test and the material that has come into contact with the sample must be decontaminated and disposed of in accordance with the laboratory procedures for the decontamination and disposal of potentially infected material.

REFERENCES

1. Farmer III J.J. et al (1985) Biochemical identification of new species and biogroups of Enterobacteriaceae isolated from clinical specimens. J. Clin. Microbiol. 21:46-76.

PRESENTATION

Product	Ref.	Tests
Citrate Test	88042	30 tubes

TABLE OF SYMBOLS

LOT Batch code	IVD In Vitro Diagnostic Medical Device	Manufacturer	Use by	Fragile, handle with care
REF Catalogue number	Temperature limitation	Contains sufficient for <n> tests	Caution, consult accompanying documents	Keep away from heat sources

LIOFILCHEM® s.r.l.

Via Scozia, Zona Ind.le - 64026, Roseto degli Abruzzi (TE) - ITALY
Tel +39 0858930745 Fax +39 0858930330 Website: www.liofilchem.net E-mail: liofilchem@liofilchem.net

Rev.1 / 11.08.2014