User Information

POLYMER-COATED POWDER FREE LATEX EXAMINATION GLOVES NON STERILE

Available size : S/7, M/8, L/9, XL/10

Manufacturer Address :

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1 - EU Type-Examination

a - This product is classed as Category I Personal Protective Equipment (PPE) according to PPE Regulation (EU) 2016/425 and has been shown to comply with this Regulation through the Harmonised European Standards EN 420:2003+A1:2009

b - This product is Class I medical equipment under Council Directive 93/42/EEC, Annex IX Classification Criteria per Rule 1 and Rule 5.

c - The EU Declaration of Conformity is accessible at www.mediprotec.com

2 - EN 420:2003+A1:2009

Clause 5.1 - Length and fit : PASS

Clause 5.2 - Dexterity : Level 5

Glove size	7 - S	8 - M	9 - L	10 - XL
Minimum length / mm	230	240	250	260

Performance level	1	2	3	4	5
Diameter of dexterity pin / mm	11.0	9.5	8.0	6.5	5.0

Gloves for Special Applications (EN 420:2003, Clause 5.1.3)

These gloves are manufactured with unique standard sleeve length. Therefore, they are not meeting the Clause 5.1.3 from rule EN420:2003, describing the sleeves length for each size of gloves. Nevertheless, as special application for these gloves is to protect fingers and palm area from liquid splashes, they can be considered as totally adapted to their special use.

3 - Product Instruction for Use

a - Usage - For Single Use only. If re-used, the risk of contamination and infection increases due to improper cleaning processes; and increased risk of holes and tear during re-use to weakening of gloves by cleaning processes.

b - Sizing - Select the right size glove for your hand

c - Donning - Prior to donning, perform hand hygiene using hand rub or antimicrobial solution with water. Hold glove by the bead with one hand. Align the glove thumb with your other hand thumb and slide your hand into the glove, one finger into each glove finger. Pull by the glove palm to get a good fit. Don the other glove by the same procedure.

d - Inspection - Punctures or tears may occur after donning. Inspect each glove after donning, and immediately discontinue use if found damaged.

e - During Use - Gloves polluted by blood or other organic fluids must be immediately removed as medical waste. In case of pricking or tearing, user should immediately change gloves. In case of feeling looseness on fingers, user should change gloves.

f - Doffing - Hold glove bead and pull toward the fingers until glove comes off.

g - Disposal - Properly disposal of all used gloves. Follow your Institution's policies for disposal.

WARNING

- This glove contains natural rubber latex which may cause allergic reactions, including anaphylactic responses. If allergic reactions occur, seek for medical advice immediately.

- For minimal risks only

4 - Handling and Storage

Store in a cool and dry place. Opened boxes should be kept away from fluorescent and sunlight. Gloves are packed in boxes which is suitable for transport. Keep the gloves in the box when not in use.

5 - Shelf life

The shelf life of product is 5 years from date of manufacture.

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