XKM RS232 10 Med

de	Montageanweisung Kommunikationsmodul
da	Monteringsanvisning kommunikationsmodul
en	Fitting instructions Communication module
es	Instrucciones de montaje Módulo de comunicación
fi	Asennusohje Tiedonsiirtomoduuli
fr	Notice de montage Module de communication
hr	Upute za ugradnju Komunikacijski modul
it	Istruzioni di montaggio Modulo di comunicazione
nl	Montagehandleiding communicatiemodule
no	Monteringsveiledning kommunikasjonsmodul
pt	Instruções de montagem Módulo de comunicação
sl	Navodila za montažo Komunikacijski modul
sv	Monteringsanvisning Kommunikationsmodul

de 4
da 15
en 26
es 37
fi 48
fr 59
hr 70
it 81
nl 92
no103
pt114
sl125
sv136

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Notes about these instructions	26
Intended use	27
Intended use.....	27
Queries and technical problems.....	27
Items supplied	28
Items supplied	28
Optional accessories.....	28
Guide to the appliance	29
Overview.....	29
Switching on and off	29
Control LEDs	29
Warning and Safety instructions	30
Fitting and installation	31
Module slot.....	31
Installation	32
Commissioning	33
Commissioning.....	33
Configuring the interface.....	33
Technical data	34
Caring for the environment	35
Disposal of the packing material	35

Important warnings

 Information which is important for safety is highlighted in a thick framed box with a warning symbol. This alerts you to the potential danger of injury to people or damage to property. Read these warning notes carefully and observe the procedural instructions and codes of practice they describe.

Notes

Information of particular importance that must be observed is highlighted in a thick framed box.

Additional information and comments

Additional information and comments are contained in a simple frame.

Operating steps

Operating steps are indicated by a black square bullet point.

Example:

- Select an option using the arrow buttons and save your choice with *OK*.

Display

Certain functions are shown in display messages using the same font as that used for the function itself in the display.

Example:

Settings  menu.

Intended use

With Miele washer-disinfectors, cleaning processes can be documented. To enable this, Miele washer-disinfectors are equipped with a module slot on the back to take a Miele communication module.

The XKM RS232 10 Med communication module supplied enables the installation of an RS 232 interface.

Only use terminal devices (printers etc.) which are approved according to EN/IEC 60950.

For more information on suitable printers please contact Miele. Configuration of the interface must only be carried out by suitably qualified and competent personnel.

The communication module can be used in the following washer-disinfectors:


- PG 8562
- PG 8581
- PG 8582
- PG 8582 CD
- PG 8583
- PG 8583 CD
- PG 8591
- PG 8592
- PG 8593

The communication module is generally referred to as the module in these Operating instructions.

Queries and technical problems

In the event of queries or technical problems, please contact Miele. Contact details can be found at the end of the operating instructions for your cleaning machine or at www.miele.com/professional.

Items supplied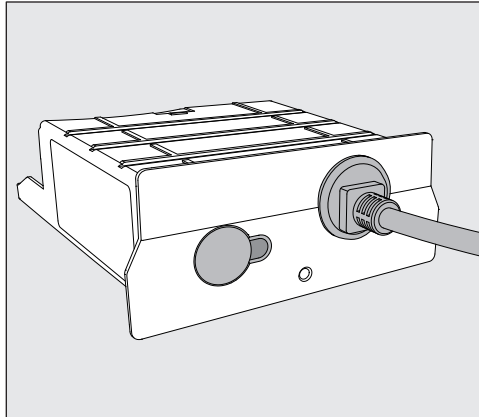


- Communication module XKM RS232 10 Med

Optional accessories

- PRT 100 report printer for printing process documentation
- Net500 network converter for converting RS 232 serial data into network data (TCP/IP), for connecting washer-disinfectors with a serial interface to a network.
- Serial Industrial Converter for serial connection of up to 2 washer-disinfectors / small steam sterilisers
- Various serial connection cables

Overview

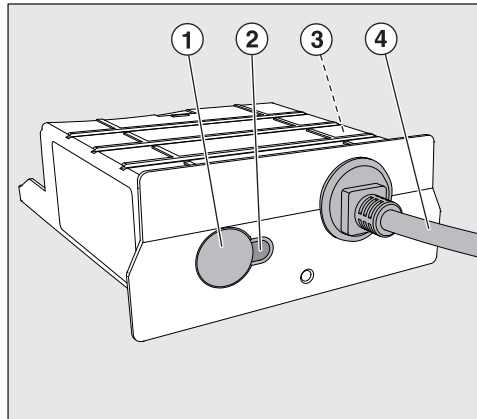- ① Push button
- ② Indicator LED,
Indicates ready-for-use status after installation and active communication during operation
- ③ Data plate (located on the side)
- ④ Data cable

Switching on and off

The module does not have a separate On/Off button. It is switched on and off in conjunction with the washer-disinfector.

Control LEDs

The control LEDs (light emitting diodes) give status information about the communication module.

LED	Status
FLASHES YELLOW	The module has connected with the terminal device and is ready for use.
LIT UP YELLOW	The module is transmitting data.
LIT UP RED	The LED lights up red when the the push button is pressed and the module is correctly installed. If the LED does not light up when the button is pressed, either the module is incorrectly installed, or the washer-disinfector is not switched on, or the module is faulty.

en - Warning and Safety instructions

To avoid the risk of accidents and damage to this communication module please read these instructions carefully before using it for the first time.

Keep these instructions in a safe place.


 Please also read the operating instructions for your washer-disinfector and pay particular attention to the Warning and Safety instructions.

- ▶ This communication module must only be used for applications specified in the operating instructions. Changes or modifications to it and incorrect use of it are not permitted as this can be dangerous.
- ▶ The communication module must only be installed, commissioned and serviced by Miele service or by a suitably qualified person. Repairs are only to be undertaken by Miele Service. Repair work by unqualified persons could be dangerous.
- ▶ Check the communication module carefully before installation for any visible damage. A damaged module could be dangerous and must not be installed.
- ▶ Before installing the communication module compare the connection data (voltage and frequency) on the data plate with that on the data plate of your washer-disinfector. If the data does not match, the module could sustain damage. If in any doubt, consult an electrician.
- ▶ The communication module housing must not be opened on any account.
Touching live electrical components or altering the electrical or mechanical components is dangerous and could cause the module to malfunction.

Module slot

The module is inserted into the module slot on the back of the washer-disinfector.

- If the washer-disinfector is installed in a niche:
 - Pull the washer-disinfector forwards out of the niche.
- If the washer-disinfector is installed under a worktop:
 - Open the door.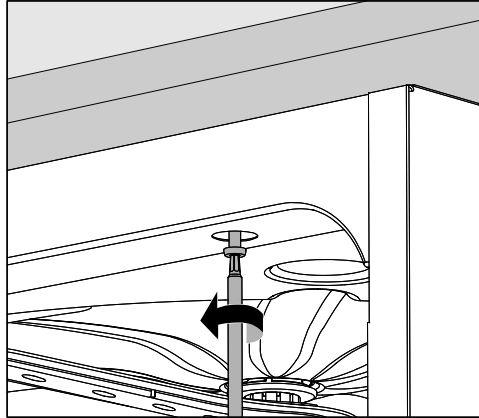


- Unscrew the fixing screws.
- Pull the washer-disinfector out from under the worktop.

Installation

⚠ Before installing the communication module compare the connection data (voltage and frequency) on the data plate with that on the data plate of your washer-disinfector. If the data does not match, the module could sustain damage. If in any doubt, consult an electrician.

- Disconnect the washer-disinfector from the mains electricity supply.
- Remove any stickers or protective caps as necessary from the slot aperture.

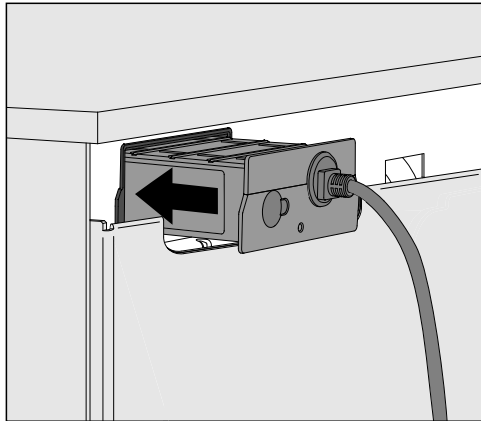- Push the module into the module slot. Gently push the module until it clicks into place.
- Establish a cable connection between the module and your terminal device.
Make sure that the cable is laid correctly and that it is not kinked or trapped anywhere. It must be protected from sharp edges and not subjected to tension.

Only use terminal devices (PC, printers etc.) which comply with EN/IEC 60950.

- Connect the washer-disinfector to the electrical supply.
The module is supplied with electricity via the washer-disinfector. An additional electrical connection is not necessary.

Commissioning

- Switch the washer-disinfector on using the  button.
- Press the push button on the module.

If the control LED lights up red, the module has been correctly installed and you can activate the serial interface and start the configuration.

If the control LED does not light up:

- The module has not been correctly installed.
- Press the module gently to check whether it has connected correctly
- The washer-disinfector is not switched on.
- Switch the washer-disinfector on using the  button.
- There may be a fault with the module
- Contact Miele Service.

Configuring the interface

Please observe the notes in the instruction manual for your washer-disinfector under “Further settings/Interface”.

en - Technical data

Voltage, Connected load, Fuse rating	See data plate
Mains cable	Approx. 1.5 m
Baud rate	9600 (ex works)
Bit	8
Parity	None (ex works)
Stop bits	1
Transfer speed	38 kBit/s
Sub-D plug pin allocation	Pin 2 RxD (input) Pin 3 TxD (output) Pin 5 GND
Temperature during operation	Up to max. 80 °C
Test certificates awarded	UL Recognised Component  us, 
CE mark	Low Voltage Directive 2006/95/EC, EMC Directive 2004/108/EC, RoHS Directive 2011/65/EU
Manufacturer's address	Miele & Cie. KG, Carl-Miele-Straße 29, 33332 Gütersloh, Germany


Disposal of the packing material

The packaging is designed to protect against transportation damage. The packaging materials used are selected from materials which are environmentally friendly for disposal and should be recycled.

Recycling the packaging reduces the use of raw materials in the manufacturing process and also reduces the amount of waste in landfill sites.

Disposing of your old appliance

Electrical and electronic appliances contain many valuable materials. They also contain certain materials, compounds and components which were essential for their correct functioning and safety. These could be hazardous to human health and to the environment if disposed of with household waste or if handled incorrectly. Please do not, therefore, dispose of your old appliance with household waste.

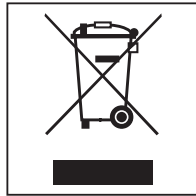Instead, please make use of officially designated collection and disposal points to dispose of and recycle electrical and electronic appliances in your local community, with your dealer or with Miele, free of charge. By law, you are solely responsible for deleting any personal data from the old appliance prior to disposal. You are legally obliged to remove any old batteries which are not securely enclosed by the appliance and to remove any lamps without destroying them, where this is possible. These must be taken to a suitable collection point where they can be handed in free of charge. Please ensure that your old appliance poses no risk to children while being stored for disposal.

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