Chloramphenicol Supplement

Selective supplement for isolation of yeasts and moulds from clinical and nonclinical specimens.

DESCRIPTION

Chloramphenicol Supplement is a selective lyophilized supplement used for the preparation of Rose Bengal Agar Base (Ref. 610178) and other culture media used for the cultivation and isolation of pathogenic and nonpathogenic fungi.

KIT CONTENTS

Each kit contains:

- 10 vials of lyophilized Chloramphenicol Supplement
- 1 instructions sheet

PRINCIPLE OF THE METHOD

Chloramphenicol is a broad-spectrum antibiotic, inhibitory to a wide range of Gram-negative and Gram-positive bacteria.

COMPOSITION

	Content / vial	Content / liter of medium
Chloramphenicol	0.05 g	0.1 g

PROCEDURE FOR USE

1. Reconstitute aseptically the content of one vial of Chloramphenicol Supplement with 7 ml of a solution containing 50% ethyl alcohol and 50% sterile distilled water.
2. Mix to complete dissolution and add to 500 ml of Rose Bengal Agar Base before sterilizing in autoclave at 121°C for 15 minutes.
3. Cool to 45-50°C, mix with care and pour aseptically into Petri dishes.

TECHNIQUE AND INTERPRETATION OF THE RESULTS

Refer to the technical sheet of the medium being prepared.

QUALITY CONTROL

1. Visual inspection: whitish button, limpid solution once reconstituted.
2. Microbiological control.
Prepare the medium per label directions. Inoculate the plates with the microbial strains indicated below and incubate at 25-30°C for 2-7 days.

Control strains		Growth	Colony color
<i>Candida albicans</i>	ATCC® 10231	Good	Pink
<i>Aspergillus brasiliensis</i>	ATCC® 16404	Good	White mycelium, black spores
<i>Escherichia coli</i>	ATCC® 25922	Inhibited	---

WARNING AND PRECAUTIONS

The product is classified as hazardous under current legislation. It is recommended to consult the safety data sheet for its correct use. The product is intended for *in vitro* Diagnostic Use only and must be used by properly trained operators.

STORAGE AND TRANSPORT CONDITIONS

2-8°C away from light, until the expiry date on the label. However, our stability studies have shown that the storage or transport at 18-25°C for 4 days, or at 35-39°C for 48 hours, do not alter in any way the performance of the product. Eliminate if signs of deterioration or contamination are evident.

REFERENCES

- Marshall, R.T. ed. (1993). Standard methods for the examination of dairy products, 16th ed. American Public Health Association, Washington, D.C.
- Koburger, J.A. (1972). Fungi in foods. Effect of pating medium pH on counts. J. Milk Food Technol. 35:659-660.
- Jarvis, B. (1973). Comparison of an improved rose bengal-chlortetracycline agar with other media for the selective isolation and enumeration of molds and yeasts in foods. J. App. Bacteriol. 36:723-727.

PRESENTATION

Product	Ref.	Content
Chloramphenicol Supplement	81017	10 vials

One vial is sufficient to prepare 500 ml of medium.

TABLE OF SYMBOLS

 Batch code	 <i>In vitro</i> Diagnostic Medical Device	 Manufacturer	 Contains sufficient for <n> tests	 Temperature limitation
 Catalogue number	 Fragile, handle with care	 Use by	 Caution, consult accompanying documents	 Do not reuse

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BIBLIOGRAFIA

- Marshall, R.T. ed. (1993). Standard methods for the examination of dairy products, 16th ed. American Public Health Association, Washington, D.C.
- Koburger, J.A. (1972). Fungi in foods. Effect of pating medium pH on counts. J. Milk Food Technol. 35:659-660.
- Jarvis, B. (1973). Comparison of an improved rose bengal-chlortetracycline agar with other media for the selective isolation and enumeration of molds and yeasts in foods. J. App. Bacterial. 36:723-727.

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LIOFILCHEM® S.r.l.