

BD BBL™ MGIT™

Mycobacteria Growth in As Little As 4–6 Days

BD BBL MGIT (Mycobacteria Growth Indicator Tube) System gives your lab a rapid, dependable, easy and safe method for growth and detection of mycobacteria from clinical specimens.

BD MGIT Fits Perfectly into Your Lab Work Flow

- No additional incubator capacity needed.
- No additional equipment required.
- No needles, no isotopes.
- No additional storage space required.

Positive for mycobacteria

Very bright orange fluorescence on tube bottom and orange reflection on meniscus.

Negative for mycobacteria

Little or no fluorescence.


Rapid,
Dependable,
Visually Distinct
Mycobacteria Detection




Indispensable to
human health




Add **BD MGIT OADC** and **BD MGIT PANTA™** to tube.

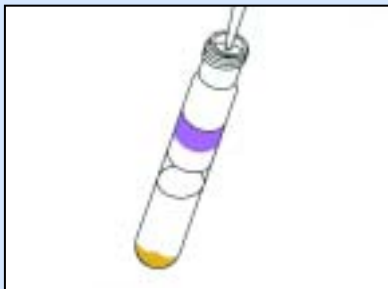

Add pretreated specimen to tube.

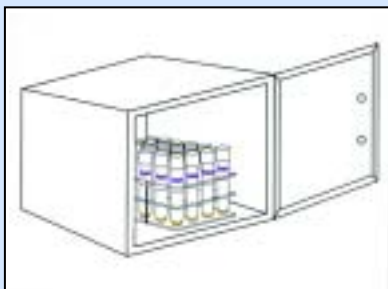

Incubate at 37°C.

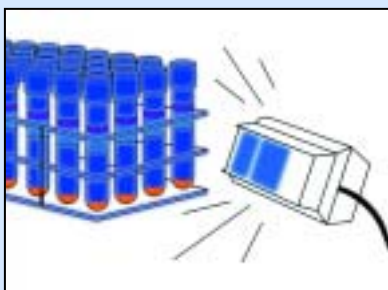

Read daily via Wood's Lamp or any long-wave UV light source beginning on day 2!*

BD BBL™ MGIT™

Easy to Use...Easy to Read...Easy to Store!

BD BBL MGIT offers you equivalent performance to the **BACTEC™ 460TB** System in a user-friendly fluorescence test format.

MGIT conveniently creates an environment suitable for rapid mycobacterial growth. You inoculate the tube with a pretreated specimen suspension and incubate. Under UV light, growth is indicated by an unmistakable orange fluorescent glow. Mycobacteria are then easily extracted for use with identification and susceptibility tests.

PRODUCT	UNIT	CATALOG NUMBER
BBL MGIT Mycobacteria Growth Indicator Tube	Box of 25 Tubes Box of 100 Tubes	245111 245113
BBL MGIT OADC Enrichment	Box of 6 Vials	245116
BBL MGIT PANTA Antibiotic Mixture (lyophilized)	Box of 6 Vials	245114
TB Fluorescent Stain Kit M (Using Auramine M Solution)	1 Kit	212519
TB Fluorescent Stain Kit T (Using Auramine-Rhodamine T Solution)	1 Kit	212521
TB Quick Stain™ Kit	1 Kit	212315
TB Stain Kit K	1 Kit	212522
TB Stain Kit ZN	1 Kit	212520
BBL MycoPrep™	1 Kit (10 x 75 mL) 1 Kit (10 x 150 mL)	240862 240863
BBL AFB QC Slide	Box of 50 Slides	231391



*Please refer to package insert for complete procedural details.

BD, Logo and all trademarks are the property of Becton, Dickinson and Company. ©2001. 5-2283 April 2001 Printed in USA

BD Diagnostic Systems
7 Loveton Circle
Sparks, MD 21152-0999
800.638.8663
www.bd.com/clinical